

# LAUNDERING PROCESS FOR CLEANING MATTRESSES AND CUSHIONS

STEP 1...

PERSONAL PROTECTIVE EQUIPMENT MUST BE WORN WHEN LAUNDERING. THIS INCLUDES BUT IS NOT LIMITED TO GLOVES, APRONS AND SLEEVES.

STEP 2...

REMOVE MATTRESS / CUSHION FROM DIRTY HOLDING AREA AND PLACE ON DIRTY BENCH.

STEP 3...

REMOVE AND DISPOSE OF PLASTIC PACKAGING.

STEP 4...

UNROLL MATTRESS ALONG BENCH.

STEP 5...

UNZIP AND REMOVE MATTRESS / CUSHION COVER.

STEP 6...

PLACE MATTRESS / CUSHION COVER INTO WASHING MACHINE.

STEP 7...

TURN MATTRESS / CUSHION UPSIDE DOWN.

STEP 8...

USING TUFFIE 5s...

WIPE DOWN ENTIRE UNDERSIDE SURFACE OF THE MATTRESS / CUSHION.

STEP 9...

CARRY MATTRESS / CUSHION TO CLEAN BENCH AND PLACE BOTTOM SIDE DOWN CAREFUL NOT TO TOUCH THE CELLS TO THE BENCH.

STEP 10...

WIPE DOWN CELLS AND INSIDE OF BOTTOM COVER WITH TUFFIE 5s.

STEP 11...

DISPOSE OF PERSONAL PROTECTIVE EQUIPMENT AND STACK MATTRESS / CUSHION FROM TOP TO BOTTOM ON DRYING RACK.

STEP 12...

WITH GLOVES ON, WIPE DOWN DIRTY BENCH WITH TUFFIE 5s. REPEAT STEPS 1-11 UNTIL DRYING RACK OR WASHING MACHINE IS FULL.

STEP 13...

PLACE ONE SCOOP OF ECOLAB ELTRA FOR EACH COVER INTO TOP 'SECTION B' OF MACHINE.

STEP 14...

PROCESSOR CONTROL  
PRIOR WASH RESULTS

MINUTES REMAINING\* 12  
PROGRAM 12  
SELECT PROGRAM  
START

PROGRAMS  
1. HOT WASH WITH PRE-RINSE  
2. RINSE WASH WITH PRE-RINSE  
3. COLD WASH WITH PRE-RINSE  
4. PERM PRESS WASH WITH PRE-RINSE  
5. HOT WASH WASH ONLY  
6. WARM WASH WASH ONLY  
7. COLD WASH WASH ONLY  
8. PERM PRESS WASH ONLY  
9. DELICATE

WFS

SET WASHING MACHINE TO PROGRAM 12 AND START CYCLE.

STEP 15...

WITH GLOVES ON, REMOVE PUMP UNIT FROM DIRTY HOLDING BAY.

STEP 16...

REMOVE AND DISPOSE OF PLASTIC PACKAGING. WIPE DOWN PUMP UNIT FROM TOP TO BOTTOM AND PLACE ON CLEAN BENCH FOLLOWED BY THE POWER LEAD.

STEP 17...

CARRY PUMP AND LEADS TO TESTING BENCH READY FOR ELECTRICAL SAFETY TESTING.

STEP 18...

ONCE THE MATTRESSES / CUSHIONS HAVE AIR DRIED, CARRY THEM TO THE TESTING BENCH FOR INSPECTION.

STEP 19...

WITH GLOVES ON, CLEAN THE RIM OF THE WASHING MACHINE.

STEP 20...

WITHOUT PERSONAL PROTECTIVE EQUIPMENT, EMPTY COVERS FROM WASHING MACHINE INTO BLACK TUB FOR TRANSPORTING.

STEP 21...

PLACE COVERS INTO DRYING MACHINE. ONCE CYCLE IS COMPLETE, BRING COVERS TO TESTING BENCH FOR THE MATTRESSES / CUSHIONS TO BE REASSEMBLED.

FINAL STEP

REPEAT ENTIRE PROCEDURE UNTIL THE DIRTY HOLDING BAY IS EMPTY. ONCE FINISHED, WIPE DOWN ALL SURFACES WITH TUFFIE 5s